



## THERAPY INSTRUCTIONS

### Step 1: Gather and Prepare equipment

<input type="checkbox"/>	Dioxyfin canister
<input type="checkbox"/>	Sprayer bottle (filled with water), hose or bucket of water and sponge
<input type="checkbox"/>	Sleeve
<input type="checkbox"/>	Stretch bandage
<input type="checkbox"/>	Vet wrap

### Step 2: Prepare The Animal For Therapy

<input type="checkbox"/>	If treating fresh wound, cover wound with two layers of gauze
<input type="checkbox"/>	If treating a horse leg or foot, wrap bottom of hoof in vet wrap
<input type="checkbox"/>	Use sprayer bottle, sponge or hose to wet entire leg surface, <b>including the skin</b> with warm water
<input type="checkbox"/>	Pull on sleeve
<input type="checkbox"/>	Use the stretch bandage to remove all air from inside the sleeve
<input type="checkbox"/>	Seal the top of the sleeve with stretch bandage or with vet wrap
<input type="checkbox"/>	Remove and store the stretch bandage

### Step 3: Perform The Therapy

<input type="checkbox"/>	Place the canister spout into the valve on the sleeve
<input type="checkbox"/>	Release Dioxyfin into the sleeve. <b>NOTE: cover should not be inflated like a balloon.</b>
<input type="checkbox"/>	Leave sleeve in place on patient for 20 minutes.

### Step 4: Post Treatment

<input type="checkbox"/>	After completing therapy, remove wrap from top of the sleeve
<input type="checkbox"/>	Remove the sleeve from the leg and discard
<input type="checkbox"/>	Store or discard the Dioxyfin canister

**Chronology of Weekly Care:**  
Recommended treatment is three consecutive days and then every other day for six additional treatments

	M	T	W	T	F	S	S
1							
2							
3							
4							
5							
6							
7							

For more information and for instructional videos visit  
[www.serendimedical.com](http://www.serendimedical.com)

